# NUK<sup>®</sup>

## BP7

### Elektrische Milchpumpe Electric Breast Pump

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# Contents

<b>1 Important Information Before First Use</b>	<b>23</b>
<b>2 Description of Device</b>	<b>23</b>
2.1 Intended use	23
2.2 Models	23
2.3 Description of function	23
2.4 Contraindications	24
2.5 Side effects	24
2.6 Symbols on the product / the packaging	24
<b>3 Safety Information</b>	<b>25</b>
3.1 Explanation of terms	25
3.2 General information	25
3.3 Hygiene	25
3.4 Notes for the Breast Milk Container and Teat	26
3.5 EMC	26
<b>4 Designation of Components and functional Elements (Figure A)</b>	<b>27</b>
<b>5 Cleaning and Sterilising</b>	<b>28</b>
5.1 Cleaning (Figure A in reverse order)	28
5.2 Sterilising	28
5.3 Special cleaning of tube and filter	29
5.4 Tube blow-dry function	29
<b>6 Assembling the pump</b>	<b>30</b>
<b>7 Power supply</b>	<b>30</b>
7.1 Mains operation	30
7.2 Power supply via a USB power source	30
7.3 Battery operation (Nature Sense model only)	30
<b>8 Operation</b>	<b>31</b>
8.1 Expressing milk	31
8.2 Automatic switch-off	33
8.3 Mobile operation	33
8.4 Feeding expressed milk	33
<b>9 Tips for Successful Expressing</b>	<b>33</b>
9.1 Frequency	33
9.2 Method	33
9.3 Quantity of milk	34
9.4 Let-down reflex	34
<b>10 Hygiene tips</b>	<b>34</b>
<b>11 Storing Milk</b>	<b>34</b>
11.1 With Breast Milk Container	34
11.2 Important notes for handling breast milk	35
<b>12 Troubleshooting</b>	<b>36</b>
<b>13 Technical Data</b>	<b>38</b>
<b>14 Maintenance</b>	<b>38</b>
<b>15 Disposal</b>	<b>39</b>
15.1 Device	39
15.2 Packaging	39
<b>16 Warranty</b>	<b>39</b>
16.1 Service Centres	39
16.2 Manufacturer	39
<b>17 EU declaration of conformity</b>	<b>39</b>

# Dear Mother,

Breast milk is the best food for a baby during the first six months of its life. To enable you and your baby to enjoy every moment of breastfeeding, NUK has developed a range of practical and comfortable breastfeeding aids for the systematic support of breastfeeding.

The NUK Breast Pump is intended as an aid for you and your baby if you are unable to

breastfeed naturally on a temporary or permanent basis. Please take a few minutes to read carefully through these operating instructions before using your NUK Breast Pump for the first time.

You can find brief instructions in the form of illustrations in figure E.

We wish you and your baby all the best.

## 1 Important Information Before First Use

**Warning!** Before using the NUK Breast Pump for the first time, clean and sterilise all parts (see “5 Cleaning and Sterilising“ on page 28).

Please carefully read the enclosed information on use and warnings for the NUK breast milk container / bottle and teat, as well as the accessories.

## 2 Description of Device

### 2.1 Intended use

The NUK electric breast pumps (of the BP7 series), available as First Choice<sup>+</sup> or Nature Sense models, are an aid for expressing and collecting breast milk:

- in the workplace or when baby is away, at home or when travelling
- to maintain milk production during a break in breastfeeding
- to relieve engorgement
- to increase milk production

Once expressed, breast milk can be collected, stored, frozen, warmed and fed to your baby in the NUK Breast Milk Container. These pumps are not certified for commercial use.

### 2.2 Models

This NUK breast pump is available as three different models:

- First Choice<sup>+</sup> model as a single pump
- Nature Sense model as a single pump
- Nature Sense model as a double pump.

**First Choice<sup>+</sup> model:** comes with a NUK First Choice<sup>+</sup> accessory package. It can be

operated using the mains adapter included in the NUK packaging, or an external USB power source.

**Nature Sense model:** comes with a NUK Nature Sense accessory package. It can be operated using the mains adapter included in the NUK packaging, a built-in rechargeable battery, or an external USB power source.

#### **Single and double pump**

The Nature Sense pump is available as a single or a double pump. The single pump contains one breast shield and breast milk container. The double pump contains two breast shields, each with a breast milk container, enabling you to express milk from both breasts simultaneously.

The basic operation is essentially the same. Any specific operational features are separately indicated by model name.

### 2.3 Description of function

This NUK Breast Pump is an electric suction pump and offers the following benefits:

- adjustable suction rhythm
- adjustable suction strength
- ergonomically-shaped breast shield

- soft silicone cushion
- operation using mains adapter included in the NUK packaging or with an external USB power source (not included in the NUK packaging)
- Nature Sense model can also be operated using the integrated rechargeable battery (see “7.3 Battery operation (Nature Sense model only)” on page 30)
- collects expressed breast milk

## 2.4 Contraindications

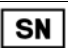
In the event of breast problems (infections, open cuts), the NUK Breast Pump should only be used after consulting your doctor or health visitor supervising treatment.

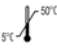
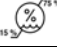
## 2.5 Side effects

If at any time you experience an unexpected reaction, please consult your midwife, health visitor, doctor or pharmacist.

Please report any serious incident in relation to the device to the manufacturer and your national competent authority (see “16.2 Manufacturer” on page 39).

## 2.6 Symbols on the product / the packaging

Symbol	Meaning
	Manufacturer
	Refer to the operating instructions
	Attention! Observe the safety instructions in the operating instructions
	Serial number
BP7	Model name
	Complies with all the requirements of applicable EC directives
	Electrically protected device, Protection Class II
	Electromedical device, Type BF

Symbol	Meaning
IP22	Protection against foreign bodies > 12.5 mm and against falling, dripping water, if the housing is inclined up to 15°
	Direct current
	Do not dispose of batteries or the pump with household waste
	CN RoHS
	Keep away from sunlight
	Temperature limit at storage +5 °C to +50 °C
	Keep dry
	Humidity limit at storage 15% - 75% rH
	Article number
	Batch identification
	Please note that the packaging can be recycled and disposed of accordingly
	Regulation (EU) 2017/745

# 3 Safety Information

## 3.1 Explanation of terms

The following keywords are used in these user instructions:

**Warning!**

Ignoring this warning could result in a serious risk to life and limb.

**Caution!**

Ignoring this warning can cause minor personal injury and/or property damage.

**Note:**

Points to be noted when handling the appliance.

## 3.2 General information

**Warning!** Keep small parts out of the reach of children: risk of swallowing and suffocation.

**Warning!** Strangulation hazard! Always keep cables, tubes and straps out of children's reach.

When not in use: Make sure the power cable of the mains adapter, tube and carrying strap are safely tucked away to prevent babies or children from strangulation and others from tripping and falling. Never place the breast pump with the tube, cable or strap inside the baby's cot, bed or playpen.

**Warning!** Never submerge the pump unit **3** in water, as there is a risk of electrocution and the motor unit might be damaged by water penetration.

**Nature Sense model:**

**Warning!** The rechargeable battery is permanently built-in and cannot be replaced by you.

An incorrectly replaced battery can result in an excessive temperature build-up, explosion and/or fire.

- Before using the pump, please carefully read the operating instructions. These instructions are part of the device and should be available at all times.

- Use the NUK Breast Pump and accessories only for their described intended use (see "2.1 Intended use" on page 23).
- Electrical medical devices such as this one must only be used by adults.
- The breast pump must only be mains-operated with the mains adapter supplied.
- Do not expose the device to direct sunlight, as UV rays and overheating may make plastics brittle.
- We recommend removing any fatty or oily creams from your breast before using the pump, as these can damage the surface of the silicone cushion.
- Use only NUK original accessories.
- Please note that any claims of liability against the manufacturer under the terms of the warranty expire if accessories are used that are not recommended in these operating instructions.
- In each case, a damaged mains adapter cable or USB cable must be replaced by the original spare part.
- In the event of any malfunction, the pump must be returned to the manufacturer for repair. Failure to do so will invalidate your warranty.
- Never open the pump unit housing **3** – **this will invalidate the warranty.**
- If the breast milk is intended for a premature baby, it is essential that you follow your doctor's advice.
- The packaging is not a toy. Keep it out of reach of children. There is a risk of injury and suffocation.

## 3.3 Hygiene

**Warning!** To ensure your baby's healthy development, hygiene is of particular importance when handling breast milk.

- To prevent infection or bacterial contamination (build-up of bacteria) and to prevent the device becoming damaged, please refer to the section entitled “5 Cleaning and Sterilising” on page 28.
- **Only a breast pump which has been thoroughly cleaned will achieve full suction strength.**
- **Before using the breast pump, wash your hands thoroughly.**
- Remove all body jewellery (e.g. piercings) before using the breast pump.

### 3.4 Notes for the Breast Milk Container and Teat

**For your child's safety and health: WARNING!**

Please carefully read the enclosed information on use and warnings for the NUK breast milk container / bottle and teat, as well as the accessories.

### 3.5 EMC

The electric breast pump complies with the limits for group 1, class B of a medical electrical device (according to CISPR 11).

Electromagnetic compatibility (EMC) refers to the ability of a device to function properly in its electromagnetic environment without inducing inadmissible electromagnetic interferences. Such interferences can be transmitted among others by connecting cables or air.

Inadmissible interference from the environment can lead to incorrect behaviour of the medical device. Likewise, in certain cases the electric breast pump can cause such interferences with other devices. To resolve the problems, one or more of the following measures should be taken:

- Change the orientation or distance of the device to the source of the interference.
- Use the electric breast pump in a different location.

- Please contact our customer service for further information.

The electric breast pump must not be used directly next to or stacked with other appliances. If such operation is required, the electric breast pump must be observed in order to check its intended operation in this arrangement.

Further precautions with regard to electrical disturbances to maintain basic safety during expected service life are not provided.

#### **Note:**

The electric breast pump has no essential features according to EN 60601-1. The system may be interfered with by other devices, even if these devices comply with the emitting requirements specified in CISPR 11 (Comité international spécial des perturbations radioélectriques - International Committee on Radio Interference).

#### **Warning!**

Portable HF communication devices (radio devices) (including their accessories, such as antenna cables and external antennas) should not be used at a distance of less than 30 cm (or 12 inches) to the parts and pipes of the electrical breast pump. Failure to do so may result in a reduction in the performance of the equipment.

#### **Warning!**

Modifications or extensions to the device or the use of non-recommended accessories (E.g. mains adapter or USB cable) can result in increased electromagnetic interference or a reduced electromagnetic immunity of the device and lead to a faulty operation.

# 4 Designation of Components and functional Elements

(Figure A)

- 1 Lug for shoulder strap
- 2 USB port for charging (cover removable)
- 3 Pump unit
- 4 Tube (2x on double pump)
- 5 Protective cap
- 6 Filter (plus 1 replacement)
- 7 L-shaped push-on tube connector (for expressing using one breast shield)
- 8 T-shaped push-on tube connector (on double pump only, for expressing using two breast shields)
- 9 Tube connector on breast shield
- 10 Breast shield (2x on double pump)
- 11 Silicone cushion (2x on double pump)
- 12 Lip valve ((2x on double pump plus 1 replacement)
- 13 Breast milk container / bottle (2x on double pump)
- 14 Mains adapter (breast pump must only be used with the mains adapter included in the NUK packaging)
  
- 15 USB cable
- 16 Feeding teat
- 17 Screw ring
- 18 Protective cap
- 19 Sealing disc
- 20 Stand
- 21 Shoulder strap

## Control buttons of the pump unit

- 22 Memory button 
- 23 Change suction rhythm 
- 24 Reduce suction strength 
- 25 Increase suction strength 
- 26 Pause button 

- 27 Master power switch 

## First Choice+ model display (LED)

The symbols light up when the appropriate function is active.

- 28 Displays the suction rhythm
- 29 Memory function  
Flashing: current pumping session is being saved  
Constant: saved pumping session is running
- 30 Pause
- 31 Blow-dry function
- 32 Displays the suction strength

## Nature Sense model display (LCD)

The symbols are displayed when the appropriate function is active.

- 33 Displays the suction rhythm
- 34 Memory function  
Flashing: current pumping session is being saved  
Constant: saved pumping session is running
- 35 Displays the expression time
- 36 Blow-dry function
- 37 Displays the charge state of the rechargeable battery
- 38 Error message !
- 39 Displays the suction strength

# 5 Cleaning and Sterilising

**Warning!** Hygiene is the top priority when handling breast milk and all parts which come into contact with it. **You should therefore only use a pump that is clean and sterilised.**

**Before using the breast pump, always wash your hands thoroughly.**

The following parts come into contact with your breast milk and must be **immediately cleaned and sterilised after every use** (see “5.1 Cleaning (Figure A in reverse order)” on page 28 and “5.2 Sterilising” on page 28):

- 10 Breast shield
- 11 Silicone cushion
- 12 Lip valve
- 13 Breast milk container
- 16 Feeding teat
- 17 Screw ring
- 18 Protective cap
- 19 Sealing disc

The following parts **must never be cleaned and sterilised with water:**

- 3 Pump unit
- 14 Mains adapter
- 15 USB cable

## 5.1 Cleaning (Figure A in reverse order)

**Warning!** To avoid the risk of electric shock:

Unplug mains adapter **14** from the wall socket before cleaning.

**Caution!** To avoid damage, carefully clean the lip valve **12**. To sterilise the lip valve, we recommend just rinsing and boiling for 5 minutes. Cleaning implements may lead to damage.

1. Dismantle the device as shown in figure **B**, proceeding in reverse from 7 to 1.

**Warning! Never** submerge the pump unit **3**, the mains adapter **14**, or the USB cable **15** in water, as this will damage the electrical contacts.

2. Clean the following parts with warm water and a little washing-up liquid (or NUK Rinsing Agent). Then rinse the parts in running water:

- 10 Breast shield
- 11 Silicone cushion
- 12 Lip valve
- 13 Breast milk container
- 16 Feeding teat
- 17 Screw ring
- 18 Protective cap
- 19 Sealing disc

**Warning!** Milk residues may damage the product and result in microbial growth.

**Caution!** The parts should not be cleaned in a dishwasher – dishwasher detergents can cause product damage.

**Note:** When rinsing with dishes (for example), certain foodstuffs may discolour the material (e.g. carrot juice) or affect its odour (e.g. fennel tea). This has no effect on the fitness for use or safety of the bottle.

3. Next, sterilise all parts (see “5.2 Sterilising” on page 28).
4. Wipe the pump unit **3** with a damp cloth.

## 5.2 Sterilising

Before sterilising, clean as described in the section entitled “5.1 Cleaning”.

**Warning!** Risk of scalding during sterilisation.

### Caution!

- For boiling or steam sterilising (temperature above 80 °C), dismantle into each of its component parts and leave unassembled, otherwise deformation can result.
- The pump unit **3**, the mains adapter **14** and the USB cable **15** must never be sterilised

in the above way, as this can damage the electrical contacts and cause deformation.

- Incorrect treatment can damage the parts.
- Before using again, allow the sterilised parts to **cool and dry completely** before reassembling.
- When sterilising, do not use any chemical disinfectants.

1. Use a NUK Steam Steriliser, following the operating instructions for the device.

Alternatively,

2. sterilise the parts in boiling water for 5 minutes.

Ensure there is sufficient water in the pan to avoid product damage.

Alternatively,

3. use a NUK Micro Express Plus Microwave Steam Steriliser for sterilisation in your microwave oven.
- Only use the NUK Micro Express Plus Microwave Steam Steriliser for microwave sterilisation. Follow the operating instructions of the device.

### 5.3 Special cleaning of tube and filter

1. If milk enters the tube **4**, switch the device off immediately.
2. Dismantle the device as shown in figure **B**, proceeding in reverse from 7 to 1.
3. Remove the filter **6** (see figure **C**).
4. Clean the following parts with lukewarm water and a little washing-up liquid and/or NUK Rinsing Agent:

- rinse filter **6** ;
- flush tube **4** with push-on tube connector **7**.

5. Reassemble according to figure **C** or **B** in reverse order and then use the blow-dry function (see “5.4 Tube blow-dry function“ on page 29) of the device to dry the tube **4** and the push-on tube connector **7**.

**Note:** Since the filter **6** would be a virtually impermeable barrier while wet or damp, it would inhibit the blow-dry function and the pump suction. Insert the dry replacement filter instead. Leave the freshly cleaned and rinsed damp filter to air-dry for at least 4 hours.

Shake the freshly cleaned filter to remove most of the water. Then leave the filter, which is still damp, to air-dry for at least 4 hours.

### 5.4 Tube blow-dry function

1. Assemble the device as shown in figure **B**. Do not insert a filter that is still damp, but use a dry replacement filter instead.
2. To start the blow-dry function press the master power switch  **27** and then simultaneously touch the buttons  **23** and  **26**.  
The blow-dry function runs for 5 minutes.
3. Check that the tube is completely dry (no droplets visible inside), before starting the next pumping session.

If the tube is not yet completely dry, start the blow-dry function again.

You can pause the blow-dry function by touching the Pause button  **26** and continue it later by touching the Pause button  **26** again.

4. To stop the blow-dry function, switch the pump unit off by pressing the master power switch  **27**.

## 6 Assembling the pump

### 6.1 To express using one breast shield

1. Assemble the device as shown in figure **B** steps 1 to 7.

**Note:** Make sure the silicone cushion **11** fits snugly in the breast shield **10** and the lip

valve **12** is properly mounted, as the pump can otherwise draw in air, which will prevent it from working properly.

### 6.2 To express using two breast shields

1. Assemble the two breast shields with the breast milk container as shown in figure **B**, steps 1 to 3.
2. Assemble the device as shown in figure **D**, steps 1 to 3.

## 7 Power supply

### 7.1 Mains operation

**Warning!** Only the mains adapter included in the NUK packaging must be used for operation from mains power.

To disconnect from the mains supply, unplug the mains adapter from the socket.

### 7.2 Power supply via a USB power source

If no mains power supply is available, you can supply the breast pump with power using a USB power source.

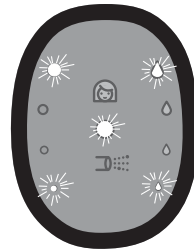
The USB power source must deliver an output current of **at least 2 amperes**.

- If the power supply supplies a current less than 2 amperes, an error message will appear in the display when the pump is switched on and you will hear a beep.

Nature Sense model display:

Symbol **!** **38**

First Choice<sup>+</sup> model display:

- No further check of the power supply is made once the device is in operation.
1. Plug the large USB connector of the USB connecting cable **15** into the USB power source.
  2. Plug the small connector of the USB connecting cable **15** into the charging socket **2** of the pump unit **3** (see step 6 in figure **B**).

### 7.3 Battery operation (Nature Sense model only)

If no mains adapter is connected, the device will run off the integrated battery. For this, the battery must be adequately charged (at least 2 bars shown in battery charge symbol), so that at least one pumping session can be completed.

The battery charge state is displayed by the battery charge symbol  **37**.

Fully charged (4 bars in the battery charge symbol **37**), the battery power will last for

three full pumping sessions of 30 minutes each.

**Note:** While connected to mains power or a USB power source, the battery will also be charged during operation.

## 8 Operation

### 8.1 Expressing milk

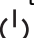
**Warning!** As hygiene is particularly important, please wash your hands thoroughly before expressing your milk, and read the information in “11 Storing Milk” on page 34.

1. Assemble the pump (see “6 Assembling the pump” on page 30).

#### Start-up mode

2. Switch on the pump unit **3** by pressing the master power switch  **27**.

This places the pump into start-up mode.

**First Choice<sup>+</sup> model:** display **30** is lit.

**Nature Sense model:** display **35** flashes (00:00) and the screen background is blue.

**Note:** The buttons **22** to **26** are touch keys. They only need to be touched lightly to select the appropriate function.

#### Starting the pumping session

##### Using one breast shield:

3. Position the breast shield **10** with the fitted silicone cushion **11** on your breast so that your nipple is in the centre of the shield. Gently press the breast shield in place in order to ensure a good seal for expressing your milk. **In doing so, take care to keep the breast milk container **13** attached to the breast shield in an upright position.**

4. To start pumping, touch either of the buttons  **23**,  **24**, or  **25**.

Pumping will start with the fastest suction rhythm and the lowest suction strength (**default setting**).

##### Using two breast shields:

5. Position both breast shields **10** with the fitted silicone cushions **11** on your breast so that your nipple is in the centre of the shield. Gently press the breast shields in place using the lower arm and hand to ensure that there is a good seal when expressing. **In doing so, keep the breast milk containers attached to the breast shields **13** as upright as possible.**

6. To start pumping, touch any of the buttons  **23**,  **24** or  **25**.

Pumping will start with the fastest suction rhythm and the lowest suction strength (**default setting**).

7. Alternatively, you can also start the pump first and select a pump setting which you have found comfortable for single-sided expressing (see “Memory function” on page 32). You then have both hands free to put on the breast shields.

**Caution!** To ensure smooth pumping, never allow the milk level to rise beyond the lower rim of the lip valve **12**.

##### Setting the suction strength

8. Use the buttons  **24** and  **25** to adjust the suction strength to one of 4 levels.

Touch-and-hold will cause the suction strength to adjust step by step up to the maximum or minimum level respectively.

The selected suction strength is displayed with **32** (First Choice<sup>+</sup>) or **39** (Nature Sense).

You can adjust the suction strength any time during the pumping session.

### Setting the suction rhythm

9. Touch the  **23** button once to adjust the suction rhythm to the next level.

Touch-and-hold will cause the suction rhythm to adjust step by step until you release the button.

The selected suction rhythm is displayed with **28** (First Choice<sup>+</sup>) or **33** (Nature Sense).

You can adjust the suction rhythm any time during the pumping session.

### Pause mode

10. Touch the Pause button  **26** to place the pump unit into pause mode. To continue pumping, touch the Pause button  **26** again.

### Memory function

The memory function can be used to save the sequence of a pumping session from the moment it is started.

11. To save a pumping session, you must first change to pause mode by touching the Pause button  **26**.

12. Then touch the Memory button  **22** for at least 2 seconds.

While the session is being saved, the symbol **29** (First Choice<sup>+</sup>) or **34** (Nature Sense) will flash.

Once the session has been saved, the pump will return to start-up mode.

13. To start the saved session, touch the Memory button  **22** while the pump unit is in start-up mode after being switched on.

The symbol **29** (First Choice<sup>+</sup>) or **34** (Nature Sense) is lit.

**Nature Sense model:** in addition, the screen background will change to red.

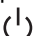
14. You can pause and resume the pumping session at any time by touching the Pause button  **26**, even while running a saved session.

15. To exit the Memory function and return to the default settings for the remainder of the pumping session, touch either of the buttons  **23**,  **24**, or  **25** while the pump unit is in pause mode.

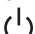
This will return your pumping session to the default settings.

### Troubleshooting

16. If you notice a reduction in the suction power over time, clean the filter **6** (see “5.3 Special cleaning of tube and filter“ on page 29), or swap the clogged filter for the replacement filter included in the NUK packaging.

17. If milk has entered the tube, immediately switch off the pump unit using the master power switch  **27**, and clean the tube and the filter **6** (see “5.3 Special cleaning of tube and filter“ on page 29).

### After each pumping session

18. Switch off the pump unit **3** by pressing the master power switch  **27**.

19. Place the breast shield **10** with the attached breast milk container **13** in the stand **20**.

20. Remove the push-on tube connector **7** from the pump unit **3**.

21. Remove the tube **4** from the breast shield **10**.

22. If you used the mains adapter **14**, unplug it in order to ensure that the device is completely disconnected from mains power.

23. Unscrew the breast milk container **13** from the breast shield **10**.

**Make sure you hold the breast milk container upright while doing so.**

24. Clean the necessary components immediately after completing your pumping session (see “5 Cleaning and Sterilising“ on page 28).

## 8.2 Automatic switch-off

If no buttons are touched during operation (during an active pumping session), the pump unit will automatically switch to start-up mode after 30 minutes.

This does not apply when the Memory function is being used. In other words, once started, a saved pumping session will always run through until the end before the device switches to start-up mode.

If no buttons are touched when the pump unit is in start-up mode or pause mode, it will automatically switch off after 15 minutes.

## 8.3 Mobile operation

You can use the pump while you are moving about, using the shoulder strap to carry it.

The shoulder strap is securely fixed to the pump unit.

1. Should the shoulder strap need cleaning, detach it from the lug **1**, wash it in water with a little soap or gentle detergent, and air dry it before reattaching it to the lug **1** on the pump unit.

**Note:** The shoulder strap **21** must not be sterilised as this might damage it.

## 8.4 Feeding expressed milk

Screw the feeding teat **16** onto the breast milk container **13** using the screw ring **17**.

If your collected breast milk is not for immediate use, firmly seal the filled breast milk container **13** with the sealing disc **19** and screw ring **17** and store it in your refrigerator or freeze it (see “11 Storing Milk“ on page 34).

# 9 Tips for Successful Expressing

You will first need to get used to your new pump. Correct expressing takes practice.

## 9.1 Frequency

How often you need to express depends on the reason for expressing. If you will be apart from your child and/or want to sustain milk production, the natural breastfeeding rhythm should be maintained. You should therefore express 6 to 8 times daily, including once during the night. In all other cases you should express entirely according to your personal wishes and requirements.

## 9.2 Method

Milk should be expressed from each breast in turn. The different settings you can choose for your pumping session allow you to emulate the natural suckling action of your baby as closely as possible.

- In the "stimulation phase", the rhythm is rapid and urgent to stimulate the milk flow reflex. This emulates the suckling action of most babies, which will initially suck rapidly and avidly at the start of feeding.
- In the "milk flow phase", suction is slow and strong to ensure maximum milk flow. Again, this corresponds to most babies' natural behaviour. As soon as a baby's initial appetite is satisfied, it will change to a calm, slow and intensive sucking action while drinking its fill.

If the milk flow drops:

- Change breasts
- Alternatively, you can repeat the "stimulation phase" in order to stimulate your milk flow reflex anew.

The amount of time needed to express milk will vary considerably from one person to another.

## 9.3 Quantity of milk

Don't be surprised if it takes a little while for the milk to start to flow properly at first. This is not unusual. In fact it is quite normal because

### **milk production adapts itself to daily requirements.**

This means that the quantity of milk produced may vary considerably.

- Ensure you drink plenty of liquid yourself.
- Do not set suction strength too high. A higher suction strength does not increase the quantity of milk produced.

## 9.4 Let-down reflex

During natural breastfeeding, placing the baby on the breast stimulates the let-down

reflex and milk begins to flow. You can also stimulate the let-down reflex as follows:

- Find yourself a quiet place to sit, where you can relax fully when using the breast pump.
- Adopt a comfortable and relaxed sitting position.
- Place a warm compress on your breast for a few minutes, to relax the tissue, and massage your breast.
- The let-down reflex triggers milk flow in both breasts simultaneously. You can use the NUK Breast Shell Set to collect breast milk leaking from the breast you are not currently expressing.

# 10 Hygiene tips

Hygiene is particularly important when using the breast pump. This applies to all the individual pump components as well as to your own personal hygiene. We recommend

regularly washing your nipples in clean water. Where possible, avoid using soap on your nipples, as this can cause irritation.

# 11 Storing Milk

## 11.1 With Breast Milk Container

Breast milk may be chilled, frozen and defrosted in the breast milk container **13**. The breast milk container **13** may also be used for warming and feeding. It is important that you follow the instructions about handling breast milk which is provided in "11.2 Important notes for handling breast milk" on page 35.

**Warning!** To ensure adequate hygiene at all times:

If your collected breast milk is not for immediate use, firmly seal the filled breast milk container **13** with the sealing disc **19** and screw ring **17** and store it in your refrigerator or freezer.

Using a suitable pen or label, write the date and the time when the breast milk was expressed. Be sure to note the retention periods in "11.2 Important notes for handling breast milk" on page 35.

**Caution!** To avoid damage:

- When freezing breast milk, do not fill the breast milk container **13** with more than 150 ml (5 fl oz), as breast milk expands as it freezes and the container could burst.

For a space saving solution you can use NUK Breast Milk Bags for freezing and storage of breast milk.

## 11.2 Important notes for handling breast milk

Breast milk	Storage location	Storage periods
Freshly expressed	At room temperature	Max. 3-4 hours
	Refrigerator (not in door)	72 hours at 4 °C
	Refrigerator freezer compartment	2 weeks
	Upright/chest freezer	6 months at -18 °C
Defrosted, unopened	Refrigerator (not in door)	24 hours at 4 °C
Defrosted, opened	Refrigerator (not in door)	12 hours at 4 °C
Defrosting milk	Gradually in refrigerator	Approx. 24 hours at 4 °C
	At room temperature followed by immediate feeding	
	If time is short, defrost container with breast milk in lukewarm water (max. 37°C), then feed immediately.	

**Warning!** To ensure adequate hygiene at all times:

If transporting chilled breast milk make sure that it stays chilled at all times, otherwise the milk could go off and be harmful to your baby. Use a cooler bag with ice packs, for example.

Defrosted, warmed or unused milk must be disposed of and not refrozen.

**Warning!** Risk of scalding and risk of destroying valuable nutrients:

Do not use a microwave to defrost breast milk, as this can destroy valuable nutrients and damage the container material (risk of explosion). Uneven heating may result in scalding!

If milk is heated excessively, you risk destroying the vitamins and other valuable nutrients it contains. There is also a risk of scalding for your baby.

Always check the temperature of your warmed up breast milk before feeding it to your baby.

1. To achieve the correct drinking temperature of approx. 37 °C, put the

container of defrosted breast milk in the baby food warmer.

**Caution!** The container must be open to prevent pressure build-up as the milk warms up.

2. Shake the milk gently before feeding so that milk and milk fat are properly mixed again.
3. If you express breast milk more than once within a period of 24 hours, the freshly expressed milk may be added to the previously cooled breast milk.

However freshly expressed milk must be cooled down before being added to milk that is already cool. The temperatures must be the same (cold on cold).

### Notes:

- Consult your health visitor, midwife, doctor or pharmacist for detailed information about long-term storage.
- You can find more information about breastfeeding on our NUK website ([www.nuk.com](http://www.nuk.com)).

## 12 Troubleshooting

Problem	Reason	Remedy
Pump does not work	No power supply	Battery operation: Check that the battery is adequately charged (see "7.3 Battery operation (Nature Sense model only)" on page 30).
		For mains operation: check that the mains plug and mains adapter are correctly inserted
		When using a USB power source: Check whether this can deliver at least 2 amperes of current.
	Lip valve missing, incorrectly positioned or damaged	Check that the lip valve is correctly positioned If it is damaged, use the replacement lip valve supplied
	Breast milk container is full and lip valve is already touching the expressed milk	Replace breast milk container
	No suction or suction level too low	Check that the lip valve is correctly fitted. Make sure that the breast shield forms a complete seal around your breast.
	Filter is wet	Remove filter and allow more time to dry or use dry replacement filter (see "5.3 Special cleaning of tube and filter" on page 29).
Pump suction is weak	Low battery power	Check whether battery power is sufficient (see "7.3 Battery operation (Nature Sense model only)" on page 30).
	Suction strength is set too low	Increase the suction strength until you achieve the best results (see "Setting the suction strength" on page 31).
	Silicone cushion incorrectly positioned and not properly flush with the breast shield	Ensure that the silicone cushion is completely flush with the breast shield
	No vacuum can be built up	Ensure that the breast shield is completely flush with the breast
	Filter blocked	Clean or replace filter (see "5.3 Special cleaning of tube and filter" on page 29).
	Filter is wet	Remove filter and allow more time to dry or use dry replacement filter (see "5.3 Special cleaning of tube and filter" on page 29).
Pump suction too strong	Suction strength is set too high	Reduce suction strength (see "Setting the suction strength" on page 31).

<b>Problem</b>	<b>Reason</b>	<b>Remedy</b>
Pump making strange noises	Lip valve unable to cope with milk flow volume	Reduce suction strength (see "Setting the suction strength" on page 31). If you have a high milk flow volume, a low suction strength will be perfectly adequate to achieve good results.  Check whether the breast milk level has already reached the lip valve. If so, replace the breast milk container with an empty one.
	Milk has entered the tube	Immediately switch off the pump unit and clean the tube and the filter (see "5.3 Special cleaning of tube and filter" on page 29).
<b>Nature Sense:</b> The display shows symbol   <b>First Choice<sup>+</sup>:</b> Symbol <b>X</b> is lit in the display	Inadequate power supply: Not enough current supplied through mains adapter or USB power source	Mains power operation: Check that the mains adapter is properly plugged into the power supply socket and the USB connector is firmly pushed into the port on the pump unit. Check that you are using the correct mains adapter included in the NUK packaging. When using a USB power source: Ensure that the USB power source used can deliver at least 2 amperes of current. If the USB power source is too weak, it cannot be used.
Replacement parts needed (filter, lip valve)		Contact your local Service Centre (see "Contact addresses" on page 186).

## 13 Technical Data

Medical product	Regulation (EU) 2017/745
NUK electric Breast Pump	complies with EN 60601-1, EN 60601-1-2, CISPR 11 Class 1 Group B
Mains adapter	complies with IEC 62368-1
NUK Teat NUK Breast Milk Container	complies with EN 14350
Suction	approx. 0 - 350 mbar
Cycles	approx. 30 - 120 cpm (cycles per minute)
Applied parts	silicone cushion
Environmental conditions	
Atmospheric pressure operation/storage/transport	70 kPa – 106 kPa
Relative humidity: operation/storage/transport	15 % – 75 %
Temperature	
• operation	+5 °C to +30 °C
• storage/transport	+5 °C to +50 °C
Power supply	
Inlet	USB-C™ compatible port (no further USB function, power supply only)
Mains power operation using mains adapter included in NUK packaging	Input: 100 - 240 V ~ / 50/60 Hz Output: 5.0 VDC, 2000 mA
USB power source (not included in NUK packaging)	Output 5 VDC, minimum 2000 mA Recommendation: power bank with at least 8000 mAh
Rechargeable battery (Nature Sense model only)	Li-ion, corresponds to IEC 62133, capacity: 2500 mAh, voltage: 3.7 VDC
Battery operating time	1.5 hours
Expected service life	at least 400 h

## 14 Maintenance

Device requires no preventive maintenance except for cleaning (see “5 Cleaning and Sterilising“ on page 28).

**Warning!** Modifications of the device are prohibited! The pump unit **3** must not be opened!

## 15 Disposal

### 15.1 Device

In the European Union, the symbol of a crossed-out wheeled bin means that the product must be disposed of at a separate waste collection point. This applies to this product and to all accessories marked with this symbol. Products labelled in this way must not be

disposed of with normal household waste, but should be taken to a collection point for recycling electric and electronic equipment.

### 15.2 Packaging

When you wish to dispose of the packaging, please observe the relevant environmental regulations for your country.

## 16 Warranty

Subject to careful treatment of your appliance and observance of these operating instructions, the manufacturer offers a 24 month warranty from the date you purchase the product.

Appliances sent to us postage paid will be repaired free of charge in the event of faulty materials or workmanship. Any further claims are in accordance with your statutory rights.

No compensation will be given for damage caused by improper use of the device.

Worn parts are excluded from the warranty.

Within the first 24 months after the date of purchase we will rectify any recognised faults without any charge for incidental costs incurred.

Under all circumstances, the device must be returned accompanied by the purchase receipt or other proof of the purchase date must be supplied.

If no proof of purchase date is provided, you will be invoiced for the costs incurred without further inquiry, and the item will be returned to you with a request of payment on delivery.

Please complete the warranty form immediately after purchase and keep this, or the till receipt, in a safe place as proof of purchase. Lost warranty certificates will not be replaced.

In case of complaints: please keep the address, Art. No., and SN for future reference. The serial number can be found on the rating plate, identified as "SN xxxx".

### 16.1 Service Centres

The addresses for service can be found under "Contact addresses" on page 186.

### 16.2 Manufacturer

Please note that the address below **is not a service address**. In the event of problems or queries relating to your product, contact one of the Service Centres (see "Contact addresses" on page 186).

MAPA GmbH  
Industriestraße 21-25  
27404 Zeven  
Germany  
[www.nuk.com](http://www.nuk.com)

## 17 EU declaration of conformity

The device meets all applicable European directives, as well as their respective essential requirements. These are detailed in the EU declaration of conformity, which can be requested from the manufacturer. The Declaration of Conformity can be found at [www.nuk.de](http://www.nuk.de).

# Warranty only valid in Australia and New Zealand

This NUK product is covered by a 24 month replacement or repair warranty, which is in addition to your rights under the Australian Consumer Law (if your product was purchased in Australia) or New Zealand Consumer Guarantees Act (if your product was purchased in New Zealand).

Upon receipt of your claim, NUK will seek to resolve your difficulties or, if the product is defective, advise you on how to obtain a replacement or refund.

In order to make a claim under our warranty, you must have the original proof of purchase documentation for the product and present it when requested.

Should your product develop any defect within 24 months of purchase because of faulty materials or workmanship, we will replace or repair it, at our discretion, free of charge. A product presented for repair may be replaced by a refurbished product of the same type rather than being repaired. Refurbished parts may be used to repair the product.

Our replacement or repair warranty only applies where a defect arises as a result of faulty material or workmanship during the warranty period. Your warranty does not cover misuse or negligent handling, accidental damage, or normal wear and tear.

Your warranty does not:

- cover freight or any other costs incurred in making a claim;
- or
- cover damage caused by: power surges, power dips, voltage supply problems, or use of the product on incorrect voltage;
  - servicing or modification of the product other than by NUK;
  - use of the product with other accessories, attachments, product supplies, parts or devices that do not conform to NUK specifications;

or

- exposure of the part to abnormally corrosive conditions;

or

- extend beyond 3 months if the product is used in commercial, industrial, educational or rental applications.

The benefits given to you by our warranty are in addition to other rights and remedies under law in relation to the product.

In Australia our goods come with guarantees that cannot be excluded under the Australian Consumer Law. You are entitled to a replacement or refund for a major failure and for compensation for any other foreseeable loss or damage. You are also entitled to have the goods repaired or replaced if the goods fail to be of acceptable quality and the failure does not amount to a major failure.

Our goods also come with guarantees that cannot be excluded under the New Zealand Consumer Guarantees Act.

If your warranty claim is not accepted, we will inform you and if requested to do so by you, repair the product provided you pay the usual charges for such repair. You will also be responsible for all freight and other costs.

Should your product require repair or service after the warranty period, contact your local NUK customer service centre.

For more information, visit our website or call our customer care line for advice on:

1800 804 918 in Australia,

or

0800 333 006 in New Zealand.

Should you experience any difficulties with your product during the warranty period, please contact NUK Customer Care.

For distributor addresses please refer to page 186.

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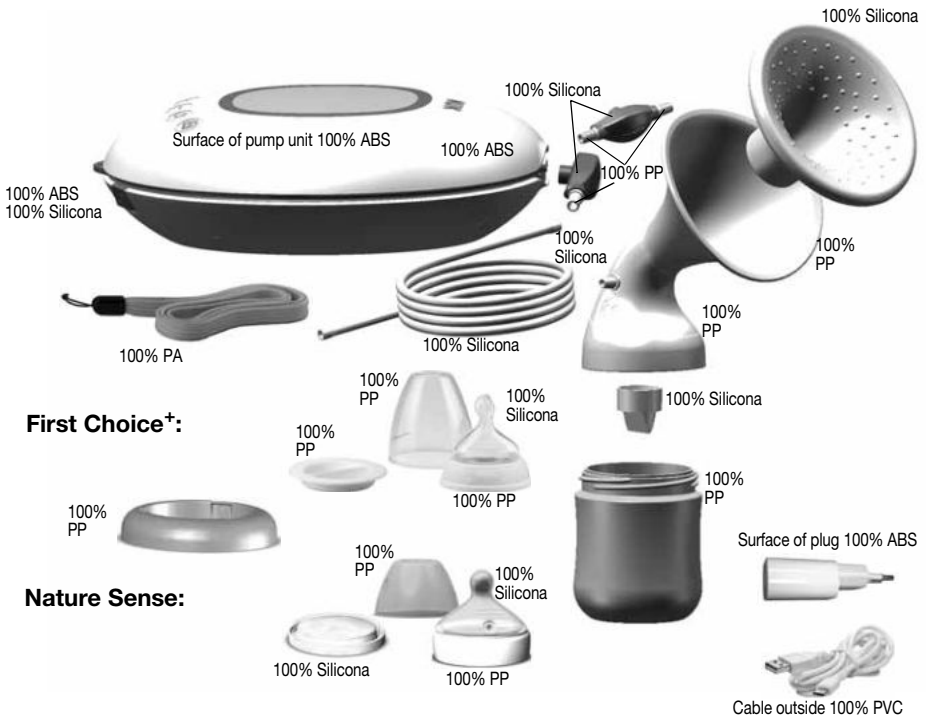
## AUS - AUSTRALIA

Newell Australia Pty Ltd  
Caribbean Park  
Level 3, 35 Dalmore Drive  
Scoresby VIC 3179  
www.nuk.com.au

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Lapsekes Oy  
Kaupinkatu 14  
45130 Kouvola  
☎ (+358) 5 3414 400

## GB - GREAT BRITAIN

Newell Rubbermaid UK Services Ltd.  
Halifax Avenue, Fradley Park  
Lichfield, Staffordshire, WS13 8SS  
www.nuk.co.uk

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## IRL - IRELAND

Intrapharma Consumer Health  
Magna Business Park,  
Citywest Road  
Dublin 24  
United Drug House  
☎ +353 1 463 2424

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## UAE - UNITED ARAB EMIRATES

Arabian Ethicals Co.  
AIRPORT ROAD, GARHOUH  
Dubai, United Arab Emirates  
☎ 04-2088400  
info@arabianethicals.ae

## ZA - SOUTH AFRICA

Artemis Brands (PTY) LTD  
P.O. Box 130782  
Bryanston 2074  
☎ +2711 430 5600  
www.nuk.co.za  
info@artemisbrands.co.za

MAPA GmbH  
Industriestraße 21-25  
27404 Zeven  
Germany  
[www.nuk.com](http://www.nuk.com)