NUK®

Baby Thermometer 2 in 1

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Congratulations on your purchase of this high-quality NUK product!

The NUK Baby Thermometer 2-in-1 will provide you with a quick and reliable temperature reading, taken at the ear or the temple. The device can also be used to determine the temperature of surfaces (e.g. bath water) and rooms.

Contents:

1	Overview.....	28
1.1	Correct use.....	28
1.2	Warning and safety instructions.....	29
1.3	Scope of supply and description.....	32
2	Use.....	33
2.1	Before initial use.....	33
2.2	Basic information.....	33
2.2.1	Fever.....	33
2.2.2	Fever thermometer.....	34
2.3	Using the thermometer.....	35
2.3.1	Measurement in the ear.....	36
2.3.2	Measurement at the temple.....	38
2.3.3	Further functions.....	39
3	Instructions for use.....	41
3.1	Changing the battery.....	41
3.2	Cleaning and storage.....	42
3.3	Disposal.....	43
4	Troubleshooting.....	43
5	Technical data.....	45
6	Warranty.....	50
7	EU declaration of conformity.....	51

1 OVERVIEW

1.1 CORRECT USE

The NUK Baby Thermometer 2-in-1 has been designed for taking a person's temperature. Using infra-red technology, the temperature can be taken at the temple or in the ear. The practical fever warning indicates an increased temperature using a red LED signal. The room temperature is shown on the display when it is idle. Temperatures of surfaces (particularly useful for testing bath water, baby food, etc.) can be simply and reliably measured by pressing the SCAN button. With its integrated battery, the NUK Baby Thermometer 2-in-1 is immediately ready for use.

The NUK Baby Thermometer 2-in-1 also has a save function for monitoring temperature progress.

The NUK Baby Thermometer 2-in-1 is solely intended for private use, and not for commercial use in medical practices or hospitals.

Please read the instructions for use in full before using the device for the first time.

Authorized representative in the European community

Battery Recycling

Indicates this device is subject to the Waste Electrical and Electronic Equipment Directive in the European Union.

To protect the environment, dispose of useless device at appropriate collection sites according to national or local regulations.

Caution

Manufacturer

IP22 Protection level for water and dust.

Medical device of type BF

Symbol for permitted temperature range when storing the device

The CE mark and Notified Body Registration Numbers, the requirement of Annex II from Medical Device Directive 93/42/EEC are met.

Paper Recycling

1.2 WARNING AND SAFETY INSTRUCTIONS

The following provides an overview of all safety instructions which must be observed to ensure the safe use of your NUK Baby Thermometer 2-in-1. In the operating instructions, these safety instructions appear in the order below, indicated by the symbol  and the relevant number. Notes on effective and optimized use of the device are marked with **Note**. In these operating instructions, steps to be completed in consecutive order are indicated by a white number in a black circle (e.g. step 1 as: ).

Warning and safety instructions	Section	 no.
The NUK Baby Thermometer 2-in-1 was developed for use by adults and young people. The device is not intended for persons with physical, sensory or mental disabilities, or those without sufficient knowledge of how to use infra-red thermometers. Children may not use the device, young people only if they have received instruction beforehand.		
The device is intended for indoor use, where possible, try to avoid using the thermometer outdoors. External influences, such as the wind and outdoor temperature, may have an impact on the measurement.	2.2.2	1
Always seek the advice of a doctor in the event of high fever or ambiguous measurements.	2.2.2	2

Warning and safety instructions	Section	 no.
Always check for any visible damage before using the device. If you discover any damage, do not use the device.	2.3	3
Check the function and accuracy of the device after any heavy impact (e.g. falling from a height of more than 1 metre) by taking several measurements. If you have any doubts concerning the measurement results, please contact your retailer.	2.3	4
Changes may only be made to the device with the consent of the manufacturer.	2.3	5
Only operate and store the device in accordance with the conditions specified.	2.3 3.2	6 20
Do not use any accessory parts that are not expressly approved for the device.	2.3	7
Do not use any additional probe covers on the device.	2.3	8
Do not hold the thermometer in your hands for longer than necessary. Otherwise, the device will warm up and may read a temperature that is too low when checking for fever.	2.3	9
If the probe head on the device glows red when the temperature is measured, continue to monitor the body temperature constantly. If in doubt about the person's condition, seek medical assistance.	2.3	10

Warning and safety instructions	Section	 no.
Take the temperature a total of three times from the same ear. If the results differ, take the highest temperature value.	2.3.1	11
Clean the device after every use in order to prevent any reinfection or infection of other persons.	2.3.1	12
The temperature of a healthy person may differ by 0.2 - 1.0°C / 0.4-1.8°F between the temple and the ear.	2.3.1	13
The person who is having their temperature taken should avoid any fluctuating ambient temperatures for at least 5 minutes before the measurement. Avoid any strenuous physical activity or taking a bath for at least 30 minutes before the measurement.	2.3.1	14
Make sure the temples are dry, clean and free from make-up when taking the temperature. Avoid placing the device on any scarred areas.	2.3.2	15
This measurement does not indicate body temperature and cannot be used for diagnosing fever.	2.3.3	16
Keep the battery out of reach of children. There is risk of ingestion.	3.1	17
The battery must be replaced as soon as the "battery empty" symbol appears in the LCD display.	3.1	18

Warning and safety instructions	Section	no. 
Do not submerge the device in water or expose it to direct moisture.	3.2	19
Do not dispose of used batteries as normal waste. Bring them to an in-store battery collection point or your nearest recycling centre for special refuse.	3.3	21
Do not dispose of the device as normal waste. Instead, take it to an official collection centre for electronic devices.	3.3	22

1.3 SCOPE OF SUPPLY AND DESCRIPTION

NUK Baby Thermometer 2-in-1 as shown in Figures 1 and 2:

- | | | | |
|---|-------------|----|---------------------------------------|
| 1 | Cover | 6 | ON/MEM button |
| 2 | LCD display | 7 | Battery compartment lid |
| 3 | SCAN button | 8 | Unlocking opening |
| 4 | Probe head | 9 | Battery CR2032 (inserted into device) |
| 5 | Sensor lens | 10 | Stand |

Operating instructions (not shown).

2 USE

2.1 BEFORE INITIAL USE

Remove all packaging material from the device, as well as the battery insulation strips. Clean the device as shown in section 3.2.

2.2 BASIC INFORMATION

2.2.1 FEVER

Fever is a state of increased body temperature.

Body temperature:	A “normal” temperature is around 36.5 to 37.0°C (97.7 – 98.6°F). The highest temperature will be read in the early evening. The values differ according to where the temperature was taken (mouth, armpit, anus, ear, temple).
-------------------	--

Causes:	Fever may result from colds, other illnesses such as respiratory infections, some intestinal infections, hyperthyroidism or the presence of tumours.
---------	--

Symptoms and diagnoses:	Sweating, shivering or chills are indications of an increased body temperature.
-------------------------	---

You should contact a doctor:	When fever starts, it may have a negative effect on the body, depending on age, the constitution and the current well-being of the person. The use of the appliance doesn't replace the investigations of the doctor. Please inform about the critical values and consult a doctor if these are exceeded.
------------------------------	---

The NUK Baby Thermometer 2-in-1 uses an infra-red sensor to measure body temperature at the eardrum membrane in the ear canal or at the skin on the temple.

Please note that when taking the temperature several times in succession from the same ear, the results may fluctuate by up to half a degree. These results may fluctuate to an even greater extent should you change to the other ear. In the event of an ear infection, damage to the ear drum or too much earwax, take the temperature at the temple instead. If the person has been lying on their ear for a longer period of time, the other ear should be used.

Note The ear canal of infants less than six months old is often very narrow, which may lead to values that are too low. It is therefore recommended that you take the temperature of infants who are less than six months old at the temple.

Note Make sure the sensor lens is clean and free from obstruction before every measurement. If this is not the case, please clean it as described in section 3.2.

1

The device is intended for indoor use, where possible, try to avoid using the thermometer outdoors. External influences, such as the wind and outdoor temperature, may have an impact on the measurement.

2

Always seek the advice of a doctor in the event of high fever or ambiguous measurements.

In the following, you will find important information regarding how the measurement results could be influenced:

	Measurement in the ear	Measurement at the temple
Reading point	At the ear drum	On the temple

	Measurement in the ear	Measurement at the temple
Factors which may influence the results	<ul style="list-style-type: none"> - The bend of the ear canal - Ear wax - Incorrect positioning of the thermometer at the ear drum - Moisture in the ear canal 	<ul style="list-style-type: none"> - Thickness of the skin at the temple - Sweat on the temple - Vasoconstrictive medication - Skin complaints - Moisturising cream on the skin

Accuracy

See section 5: Technical data

2.3 USING THE THERMOMETER

Always check for any visible damage before using the device. If you discover any damage, do not use the device.

Check the function and accuracy of the device after any heavy impact (e.g. falling from a height of more than 1 metre) by taking several measurements. If you have any doubts concerning the measurement results, please contact your retailer.

Changes may only be made to the device with the consent of the manufacturer.

Only operate and store the device in accordance with the conditions specified.

Do not use any accessory parts that are not expressly approved for the device.


Do not use any additional probe covers on the device.

Do not hold the thermometer in your hands for longer than necessary. Otherwise, the device will warm up and may read a temperature that is too low when checking for fever.

Coloured light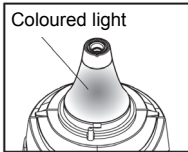

If the temperature taken by the NUK Baby Thermometer 2-in-1 exceeds or equals 37.5°C (99.5°F), a red light will appear in the probe head to warn the user of a possible fever. If the temperature taken is below 37.5°C (99.5°F), a green light will appear.

If the probe head on the device glows red when the temperature is measured, continue to monitor the body temperature constantly. If in doubt about the person's condition, seek medical assistance.

Note

The device must stay in stable ambient (room) temperature for 15 minutes before operating.

2.3.1 MEASUREMENT IN THE EAR

Take the temperature a total of three times from the same ear. If the results differ, take the highest temperature value.

Clean the device after every use in order to prevent any reinfection or infection of other persons.

The temperature of a healthy person may differ by 0.2 - 1.0°C / 0.4 - 1.8°F between the temple and the ear.

The person who is having their temperature taken should avoid any fluctuating ambient temperatures for at least 5 minutes before the measurement. Also avoid any strenuous physical activity or taking a bath for at least 30 minutes before the measurement.

1

Press the ON/MEM button () to switch on the device.

The  symbol will appear on the LCD display, and a beep will sound twice.

Note

The mode: Taking the temperature at the ear is the standard mode after switching on the device.

2

Remove the cover from the probe head.

Figure 3

3

Gently pull back the ear to lessen the bend of the ear canal.

Figure 4

4

Carefully guide the probe head into the ear canal until it rests on the ear canal.

Figure 5
Figure 6

5

Press and hold the SCAN button until you hear a beep.

6

Remove the probe head from the ear canal. Once the beep has sounded, the hourglass symbol will flash for approx. 4 seconds and the temperature taken in the ear will be shown on the LCD display. The hourglass symbol disappears again with two beeps.

7

The device automatically switches to room temperature mode after approx. one minute.

Make sure the temples are dry, clean and free from make-up when taking the temperature. Avoid placing the device on any scarred areas.

1

The device needs to be turned on first. Press ON/MEM button. Then switch to temple measurement mode. Press and hold the ON/MEM button and, at the same time, press the SCAN button once. The temples symbol will appear on the LCD display . The temple measurement mode is thus activated, used for reading the temperature at the temples.

2

Remove the cover from the probe head.

Figure 3

3

Touch the skin on the right or left temple with the probe head of the thermometer.

Figure 7

4

Briefly press the SCAN button and gently move the probe head in a circular motion over the skin on the temple. As you are doing this, you will hear a sound that indicates the temperature is being read.

Figure 8

5

A second beep sounds to indicate that the temperature has been taken.

The hourglass symbol  stops flashing after approx. 4 seconds.

Note

The time required to take the temperature ranges from 5 to a maximum of 8 seconds. This depends on how long the device needs to determine the correct temperature of the temples.

6

The temperature taken is shown on the LCD display.

7

The device automatically switches to room temperature mode after approx. one minute.

2.3.3

FURTHER FUNCTIONS

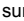
Room
temperature

After switching mode, the room temperature will be shown on the LCD display. The temperature symbol  appears.

Surface
temperature

With the device switched on, press and hold the ON/MEM button while, at the same time, briefly pressing the SCAN button several times in succession. The measurement setting will switch between ear, temple and surface temperature. Either   or  will appear on the LCD display. In the  mode, the surface temperature of objects can be determined. To take a measurement, hold the thermometer as near as possible to the object in question. The further away you are, the larger the measured area and the greater possibility of inaccuracies.

Surface
temperature

Briefly press the SCAN button to carry out a measurement.

Press and hold the SCAN button to constantly update the temperature value.

This mode is suitable for measuring the temperature of water, milk, clothing, skin or other objects.

This mode only shows the surface temperature (temperature emitted by the surface).

16

This measurement does not indicate body temperature and cannot be used for diagnosing fever.

Previous
measurement

This function is valid in ear and temple measurement mode. For every new temperature measurement, the result from the previous measurement will be shown for comparison at the top of the LCD display.

Memory (25
entries) 

With the device switched on, repeatedly press the ON/MEM button to display up to 25 previous measurements. The memory is represented by the  symbol in the LCD display.

Switching
between °C and
°F


Changing the temperature display from °C to °F:

With the device in room temperature mode, press and hold the SCAN button while, at the same time, pressing the ON/MEM button for 3 seconds.

The characters on the LCD display change: "°C" to "°F".

The same procedure applies when switching from °F to °C.

Silent mode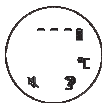

With the device switched on, press and hold the ON/MEM button for 3 seconds. The silent mode symbol  starts to flash on the LCD display. Silent mode is activated by releasing the ON/MEM button. There will be no more sounds made. The same procedure applies to deactivate silent mode.

OFF mode

To switch off the thermometer, press and hold the ON/MEM button until "OFF" appears in the LCD display.

3 INSTRUCTIONS FOR USE

3.1 CHANGING THE BATTERY

17

Keep the battery out of reach of children. There is risk of ingestion.

18

The battery must be replaced as soon as the following symbol appears in the LCD display:

In this case, proceed as follows:

1

Take a small, thin object (e.g. a paper clip) and press it into the unlocking opening of the battery compartment lid while, at the same time, pushing this opening outwards with both thumbs.

Figure
9

2

Firmly holding the device, carefully lift out the battery using a small, thin object (e.g. a paper clip).

Figure
10

3

Insert the new CR2032 battery with the positive terminal facing upwards and the minus terminal facing downwards by sliding the battery underneath the metal hook and pressing down until it clicks into place.

Figure
11

Note

If incorrectly fitted, the battery compartment and even the device may become damaged.

4

Push the battery compartment lid back onto the device until it audibly clicks into place.

3.2 CLEANING AND STORAGE

The probe head is the most sensitive part of the NUK Baby Thermometer 2-in-1. Treat this part of the device with particular care when cleaning the sensor lens.

Do not submerge the device in water or expose it to direct moisture.

After every measurement, use a cotton bud dipped in a solution of approx. 70% alcohol (Ethanol) to clean both the probe head and the sensor lens. Make sure here that the tip of the device is pointing downwards so that none of the cleaning fluid enters the housing. Then leave the probe head to dry completely for at least one minute.

We recommend that you retain the packaging of the NUK Baby Thermometer 2-in-1 so that the device can be packaged securely for transport.

If the device is not being used for an extended period of time, we recommend switching it off via the OFF mode (see section 2.3.3 OFF mode).

Store the device in a cool, dry place out of direct sunlight.

Only operate and store the device in accordance with the conditions specified.

3.3 DISPOSAL

The packaging of the NUK Baby Thermometer 2-in-1 should be disposed of in an environmentally sound manner using the relevant recycling systems.

21

Do not dispose of used batteries as normal waste. Bring them to an in-store battery collection point or your nearest recycling centre for special refuse.

22


Do not dispose of the device as normal waste. Instead, take it to an official collection centre for electronic devices.

GB

4 TROUBLESHOOTING

Error message	Problem	Possible cause	Solution
	The device is not working properly.	Software error	Remove the battery, wait one minute and then replace the battery (sect. 3.1). If the error message appears again, contact your retailer.
	While the symbol is flashing, a new measurement cannot be started in the ear/temple mode.	The device is not yet ready to take a new measurement	Wait until the symbol has stopped flashing. When it has stopped flashing, then start the new measurement.

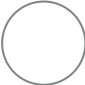Error message	Problem	Possible cause	Solution
AbH	Room temperature greater than 40.0°C (104.0°F).	The device was not stored as instructed	Place the thermometer in a room with a temperature of between 10.0°C (50.0°F) and 40.0°C (104.0°F) for at least 30 minutes.
AbL	Room temperature less than 10.0°C (50.0°F).		
H_i	(1) <i>In ear/temple mode:</i> Measurement temperature > +42.2°C (108.0°F) (2) <i>Surface temperature:</i> Measurement temperature > +80.0°C (176.0°F)	Object to be tested outside the specified measurement range (too hot)	Take a measurement within the specified temperature range. In the event of a malfunction: Contact your retailer.
Lo	(1) <i>In ear/temple mode:</i> Measurement temperature < +34.0°C (93.2°F) (2) <i>Surface temperature:</i> Measurement temperature < -22.0°C (-7.6°F)	Object to be tested outside the specified measurement range (too cold)	
	The device will not switch on and display anything.	Insulation strips have not been removed from the battery	Remove the insulation strips from the battery.
		Battery empty	Replace the battery (sect. 3.1).

5 TECHNICAL DATA

Temperature range:

Ear and temple mode:	+34.0~42.2°C (93.2~108.0°F)
Surface temperature mode:	-22.0~80.0°C (-7.6~176.0°F)
Operating temperature:	+10.0~40.0°C (50.0~104.0°F), 15%~85% RH
Storage temperature:	-20.0~+50.0°C (-4.0~122.0°F)
	Relative humidity \leq 85%
Transport temperature:	<70.0°C (158.0°F)
	Relative humidity \leq 95%

Accuracy:

Ear mode:	$\pm 0.2^{\circ}\text{C}$ / 0.4°F in area, 35.5~42.0°C (95.9~107.6°F) $\pm 0.3^{\circ}\text{C}$ / 0.5°F outside this area
-----------	--

Clinical Bias: -0.5~ -0.7 °C
Limits of Agreement: 1.02
Repeatability: 0.19°C

Temple mode:	$\pm 0.2^{\circ}\text{C}$ / 0.4°F in area, 35.0~42.0°C (95°~107.6°F), $\pm 0.3^{\circ}\text{C}$ / 0.5°F outside this area
--------------	--

Clinical Bias: -1.3 ~ -1.5 °C
Limits of Agreement: 1.47
Repeatability: 0.22°C

Surface temperature mode:	$\pm 0.3^{\circ}\text{C}$ / 0.5°F in area 22.0~42.2°C (71.6~108.0°F), otherwise $\pm 4\%$ (however at least $\pm 2.0^{\circ}\text{C}$ / 4.0°F)
---------------------------	---

Atmospheric pressure:	800~1013 hPa
Protection class:	BF
Degree of protection:	IP22
Battery:	1 CR2032 lithium battery
Battery life:	approx. 3,000 measurements (~1 year)
Comply with:	ASTM E1965-98, EN ISO 80601-2-56, IEC/EN60601-1-2(EMC), IEC/EN60601-1(Safety) standards, ISO10993, RoHS.

Note The NUK Baby Thermometer 2-in-1 converts the ear/temple temperature into its “oral equivalent” (referring to the results of the clinical assessment).

Note The lot and serial number of the device can be found on the inside of the battery cover.

Guidance and manufacturer’s declaration – electromagnetic emissions

The THixyz series is intended for use in the electromagnetic environment specified below. The customer or the user of the THixyz series should assure that it is used in such an environment.

Emissions test	Compliance	Electromagnetic environment – guidance
RF emissions CISPR 11	Group 1	The THixyz series uses RF energy only for its internal function. Therefore, its RF emissions are very low and are not likely to cause any interference in nearby electronic equipment.
RF emissions CISPR 11	Class B	The THixyz series is suitable for use in all establishments, including domestic establishments and those directly connected to the public low-voltage power supply network that
Harmonic emissions IEC 61000-3-2	Not applicable	

Guidance and manufacturer's declaration – electromagnetic emissions

Voltage fluctuations/
flicker emissions
IEC 61000-3-3

Not applicable

supplies buildings used for domestic purposes.

Guidance and manufacturer's declaration – electromagnetic immunity

Immunity test	IEC 60601 test level	Compliance level	Electromagnetic environment – guidance
Conducted RF IEC 61000-4-6	3 Vrms 150 kHz to 80 MHz	Not applicable	<p>Portable and mobile RF communications equipment should be used no closer to any part of the THixyz series, including cables, than the recommended separation distance calculated from the equation applicable to the frequency of the transmitter.</p> <p>Recommended separation distance</p> $d = 1,2 \sqrt{P}$ $d = 1,2 \sqrt{P} \quad 80 \text{ MHz to } 800 \text{ MHz}$ $d = 2,3 \sqrt{P} \quad 800 \text{ MHz to } 2,5 \text{ GHz}$ <p>where P is the maximum output power rating of the transmitter in watts (W) according to the transmitter manufacturer and d is the recommended separation distance in meters (m).</p>
Radiated RF IEC 61000-4-3	3 V/m 80 MHz to 2,5 GHz	3 V/m	<p>Field strengths from fixed RF transmitters, as determined by an electromagnetic site survey, a should be less than the compliance level in each frequency range b.</p> <p>Interference may occur in the vicinity of equipment marked with the following symbol: </p>

Guidance and manufacturer's declaration – electromagnetic immunity

NOTE 1 At 80 MHz and 800 MHz, the higher frequency range applies.

NOTE 2 These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects and people.

- a. Field strengths from fixed transmitters, such as base stations for radio (cellular/cordless) telephones and land mobile radios, amateur radio, AM and FM radio broadcast and TV broadcast cannot be predicted theoretically with accuracy. To assess the electromagnetic environment due to fixed RF transmitters, an electromagnetic site survey should be considered. If the measured field strength in the location in which the THixyz series is used exceeds the applicable RF compliance level above, the THixyz series should be observed to verify normal operation. If abnormal performance is observed, additional measures may be necessary, such as reorienting or relocating the THixyz series.
- b. Over the frequency range 150 kHz to 80 MHz, field strengths should be less than 3 V/m.

Guidance and manufacturer's declaration – electromagnetic immunity

Immunity test	IEC 60601 test level	Compliance level	Electromagnetic environment – guidance
Electrostatic discharge (ESD) IEC 61000-4-2	6 kV contact 8 kV air	6 kV contact 8 kV air	Floors should be wood, concrete or ceramic tile. If floors are covered with synthetic material, the relative humidity should be at least 30 %.
Electrical fast transient/burst IEC 61000-4-4	2 kV for power supply lines 1 kV for input/output lines	Not applicable	Mains power quality should be that of a typical commercial or hospital environment.
Surge	1 kV line(s) to line(s)	Not	Mains power quality should be

Guidance and manufacturer's declaration – electromagnetic immunity			
IEC 61000-4-5	2 kV line(s) to earth	applicable	that of a typical commercial or hospital environment.
interruptions and voltage variations on power supply input lines IEC 61000-4-11	<5 % UT (>95 % dip in UT) for 0,5 cycle 40 % UT (60 % dip in UT) for 5 cycles 70 % UT (30 % dip in UT) for 25 cycles <5 % UT (>95 % dip in UT) for 5 sec	Not applicable	Mains power quality should be that of a typical commercial or hospital environment. If the user of the THixyz series requires continued operation during power mains interruptions, it is recommended that the THixyz series be powered from an uninterruptible power supply or a battery.
Power frequency (50/60 Hz) magnetic field IEC 61000-4-8	3 A/m	3 A/m	Power frequency magnetic fields should be at levels characteristic of a typical commercial or hospital environment.
NOTE UT is the a.c. mains voltage prior to application of the test level.			

Recommended separation distances between portable and mobile RF communications equipment and the ME EQUIPMENT or ME SYSTEM

The THixyz series is intended for use in an electromagnetic environment in which radiated RF disturbances are controlled. The customer or the user of the THixyz series can help prevent electromagnetic interference by maintaining a minimum distance between portable and mobile RF communications equipment (transmitters) and the THixyz series as recommended below, according to the maximum output power of the communications equipment.

Rated maximum output power of transmitter W	Separation distance according to frequency of transmitter m		
	150 kHz to 80 MHz $d = 1,2 \sqrt{P}$	80 MHz to 800 MHz $d = 1,2 \sqrt{P}$	800 MHz to 2,5 GHz $d = 2,3 \sqrt{P}$
0,01	N/A	0,12	0,23
0,1	N/A	0,38	0,73
1	N/A	1,2	2,3
10	N/A	3,8	7,3
100	N/A	12	23

For transmitters rated at a maximum output power not listed above, the recommended separation distance d in meters (m) can be estimated using the equation applicable to the frequency of the transmitter, where P is the maximum output power rating of the transmitter in watts (W) according to the transmitter manufacturer.

NOTE 1 At 80 MHz and 800 MHz, the separation distance for the higher frequency range applies.

NOTE 2 These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects and people.

6 WARRANTY

In addition to the seller's warranty obligations arising from the contract to purchase, we as the manufacturers provide a warranty of 24 months from purchase of the device as long as the device is used properly and the instructions for use are followed. Evidence of the date of purchase and the type of device should be provided by means of a receipt. The purchaser's statutory rights are unaffected by this manufacturer warranty.

During the warranty period, we undertake to remedy any defects attributable to poor materials or workmanship. Wear parts are excluded from the warranty. Minor deviations from specified quality which do not affect the value or fitness for use of the device are not

covered by the warranty. We are likewise unable to accept the warranty if the defects on the device are due to transport damage for which we are not responsible, or, if they are attributable to misuse or lack of care or if the device has been interfered with by persons not authorized by us to do so.

The warranty service will involve repair, replacement of parts or replacement of the device at our discretion. The performance of services under warranty neither extends nor restarts the warranty period. The warranty period for any spare parts fitted will expire with the warranty period for the device as a whole. Extended or different claims, especially those relating to the making good of damage sustained outside the device, are excluded unless liability is compulsory in law.

We will not accept the costs or risks of transport. If a device is sent in without evidence of the purchase date, it will be treated as a repair case. The device will only be repaired following discussion with the customer. Please keep the article number and the lot number for future reference.

These warranty conditions apply to devices bought in Germany. The warranty conditions applicable in the country in question apply to devices not bought in Germany.

Service Centres

The addresses for service can be found under "Contact addresses" on page 96/97.

Article no.: 10.256.345

7 EU DECLARATION OF CONFORMITY

The device meets all applicable European directives, as well as their respective essential requirements. These are detailed in the EU declaration of conformity, which can be requested from the manufacturer. The declaration of conformity can be found at www.nuk.de.

The Baby Thermometer 2-in-1 meets the requirements of the standard DIN EN 12470-5 "Clinical thermometers, Part 5: Performance of infra-red ear thermometers (with maximum device)."

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NUK is a registered trademark of
Radiant Innovation Inc.
Medical Technology Promedt Consulting GmbH
MAPA GmbH
MAPA GmbH, Germany